

a perfect fit®



# CAMLOG® ALTAPIN SET MEMBRANE FIXATION

- Autoclavable set
- Clearly arranged ALTApin products
- Smooth handling
- Possibility of storing the ALTApin magazine in the set
- Two-piece tray



camlog

# CAMLOG® ALTAPIN SET MEMBRANE FIXATION

## ALTApin SET

- Includes tray with content

Resterilization of opened ALTApin magazines is possible in the provided cutout of the ALTApin tray. For that purpose, take the ALTApin magazine out of the deep-drawn component of the primary packaging.



## CONTENT OF THE ALTApin SET:

M5100.0010 ALTApin applicator, straight, incl. activator



M5100.0050 ALTApin pricker



M5100.0070 ALTApin membrane fixator



M5200.0055 ALTApin pricker, insert



M1000.0050 ALTApin magazine, 7 titanium pins,  
double sterile packaging,  
1 unit (4 ea. contained in the ALTApin Set)



## ALTApin MAGAZINE:



### Illustration:

ALTApin magazine in the deep-drawn component of the primary packaging. Secondary packaging and covering box are removed.

- ALTApin magazine charged with 7 titanium pins (pin material: Ti6Al4V)
- double sterile packaging
- non used titanium pins can be re-sterilized in the ALTApin magazine in the tray

**APPLICATION:****Pick up the titanium pin**


Set the work element of the applicator on the titanium pin head from vertically above, press slightly so that the titanium pin head gets pressed into the work element.

**Fix the membrane**

with the membrane fixator.

**Pre-prick the cavity**

with the pricker (in soft bone).

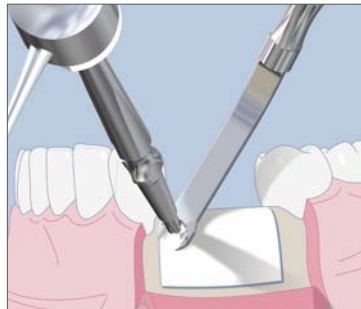**Alternatively, drill the cavity****Place the titanium pin**

with the straight applicator and the surgery mallet.

**Alternatively, place the titanium pin**

with the applicator angled 90° with finger rest.

**Remove the membrane fixator.****Fix the titanium pin**

with the blade of the pricker. Afterwards, pull off the applicator sideways.

**Membrane completely fixed.****OPTIONALLY AVAILABLE ARTICLES:**

(not part of the ALTApin Set)


M5100.0030 ALTApin applicator, angled 90°,  
incl. activator

M5200.0010 ALTApin applicator, straight, work element,  
incl. activator

M5100.0100 ALTApin surgical mallet  
(no storage space available in the ALTApin tray)

M5500.0050 ALTApin single-patient drill, ISO-shaft

M1000.0100 ALTApin magazine, 7 titanium pins, double sterile packaging,  
3 pieces (without illustration)

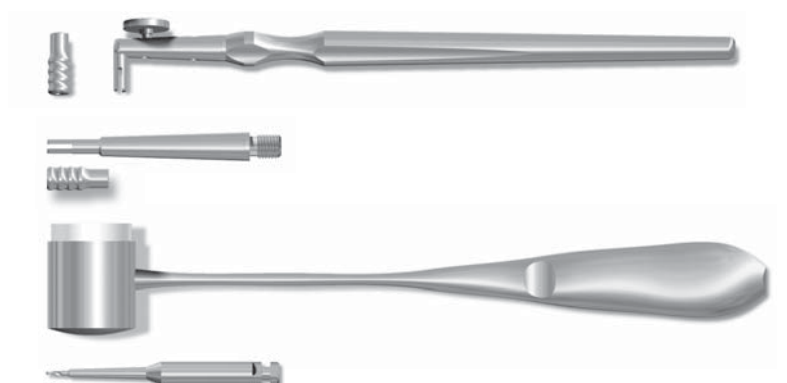

<b>ART. NO.</b>	<b>ARTICLE</b>
M5600.0110	ALTApin Set Membrane fixation system Content (also available separately): M5100.0010 ALTApin applicator, straight M5100.0050 ALTApin pricker (holder) M5100.0070 ALTApin membrane fixator M5100.0055 ALTApin pricker, insert, 2 units M1000.0050 ALTApin magazine, 7 titanium pins, double sterile, 1 units (4 ea. contained in the ALTApin Set)
M5600.0210	ALTApin tray (without content)
M5100.0030	ALTApin applicator, angled 90°, incl. activator
M5200.0010	ALTApin applicator, straight, work element incl. activator
M5100.0100	ALTApin surgery mallet*
M5500.0050	ALTApin single-patient drill, ISO-shaft
M1000.0100	ALTApin magazine, 7 titanium pins, double sterile packaging, 3 units

\* no storage space available in the ALTApin tray

## HEADQUARTERS

CAMLOG Biotechnologies AG | Margarethenstrasse 38 | CH - 4053 Basel  
info@camlog.com | www.camlog.com

Manufacturer of CAMLOG® products: ALTATEC GmbH, Maybachstraße 5, D-71299 Wimsheim

**camlog**