

THE CONELOG® IMPLANT SYSTEM – FACTS AND FIGURES AT A GLANCE

Excellent results of the CONELOG® Implant System: precision of fit and maintenance of crestal bone level.⁽¹⁾

Aim

To provide insights into the scientific documentation of the CONELOG® Implant System based on facts and figures.

Introduction

Very few implant systems have been systematically and thoroughly documented in the literature. The CONELOG® Implant System belongs to these well documented systems because encouraging independent research is fundamental to the CAMLOG strategy (**Fig. 1**).

Precision of the conical connection

CONELOG® implants offer an implant-abutment connection with self-locking cone geometry. Several mechanical tests have demonstrated the precision of the connection.^(2, 3, 4)

Microgaps and its consequence, i.e. micro-leakage or bacterial penetration in a conical connection are impossible to eliminate.^(5, 6) The lack of a micro-gap could result in cold welding of the connection and would make later replacement of the abutment almost impossible. Therefore, small tolerances are required to minimize but not eliminate this gap. The rotational freedom and ability to vertically reposition the abutment play a major role in the precision of the prosthetic restoration. An in-vitro study with hand-tightening of the abutment showed excellent results for the CONELOG® Implant System compared to 5 other systems with conical connections⁽⁷⁾ (Tab. 1).

Excellent results for the bone level changes with Platform Switching

There are several ongoing clinical studies which aim primarily to evaluate crestal bone preservation at the implant or to evaluate the outcomes of different implant lengths. Preliminary results have demonstrated good preservation of the crestal bone post loading (mean value +0.12 mm after one year⁽⁸⁾). These preliminary results confirm the outcomes of the ongoing multicenter study performed on CAMLOG® Implants with and without Platform Switching⁽⁹⁾, which demonstrated an excellent maintenance of the crestal bone level with a bone-level change at one-year post loading of +0.08 mm (with Platform Switching).

Conclusion

The solid documentation of the CAMLOG® and CONELOG® Implant Systems is based on independently collected data or on scientific evidence sponsored by the company. This is an important contribution to CAMLOG's success story. More than 11,000 implants with a Promote® surface present follow-up data of at least 5 years. The use of Platform Switching^(8, 10) and of the implant abutment connection are key factors contributing to the good integration of CONELOG® Implants.

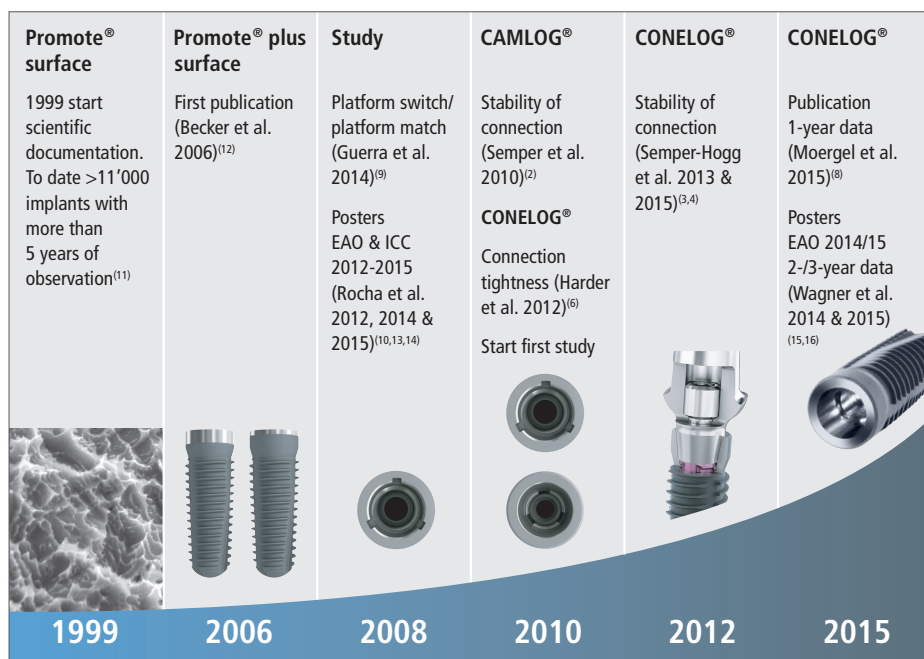


Fig 1: The development of the CONELOG® Implant System is based on a solid foundation of scientific research.

TAKE HOME MESSAGE:

1. CONELOG is a thoroughly documented implant system
2. Superior precision of the implant-abutment connection
3. Excellent maintenance of crestal bone level

Implant system	Rotational freedom (°)	Vertical height deviation (µm)
Nobel Active	> 5	> 50
Ankylos C/X	> 5	< 50
Astra Tech	> 4	> 50
CONELOG	< 3	< 50
Bone Level	> 3	> 50
Tissue Level	> 3	< 50
External hexagon impact connection (Steri-Oss, Nobel Biocare)	> 3	< 10

Tab 1: Rotational freedom (°) and vertical repositioning of 6 systems (adapted from Nelson et al. 2013)

REFERENCES

- (1)** Peters F. Wissenschaftliche Dokumentation des CONELOG® Implantatsystems – Zahlen und Fakten auf einen Blick. Logo 2015;35:6-7
- (2)** Semper W, Heberer S, Mehrhof J, Schink T, Nelson K. Effects of repeated manual disassembly and reassembly on the positional stability of various implant-abutment complexes: an experimental study. Int J Oral Maxillofac Implants 2010;25:86-94
- (3)** Semper-Hogg W, Kraft S, Stiller S, Mehrhof J, Nelson K. Analytical and experimental position stability of the abutment in different dental implant systems with a conical implant-abutment connection. Clin Oral Investig 2013;17:101
- (4)** Semper-Hogg W, Zulauf K, Mehrhof J, Nelson K. The influence of torque tightening on the position stability of the abutment in conical implant-abutment connections. Int J Prosthodont 2015;28(5): 538-541
- (5)** Harder S, Dimaczek B, Açil Y, Terheyden H, Freitag-Wolf S, Kern M. Molecular leakage at implant-abutment connection - in vitro investigation of tightness of internal conical implant-abutment connections against endotoxin penetration. Clin Oral Invest 2010; 14(4): 427-32
- (6)** Harder S, Quabius ES, Ossenkop L, Kern M. Assessment of lipopolysaccharide microleakage at conical implant-abutment connections. Clin Oral Investig 2012;16:1377-84
- (7)** Nelson K, Zabler S, Wiest W, Schmelzeisen R, Semper- Hogg W. Die Implantat-Abutment-Verbindung. Implantologie 2013;21(4):355-63
- (8)** Moergel M, Rocha S, Messias A, Nicolau P, Guerra F, Wagner W. Radiographic evaluation of conical tapered platform-switched implants in the posterior mandible: 1-year results of a two-center prospective study. Clin Oral Impl Res 00, 2015;1-8
- (9)** Guerra F, Wagner W, Wiltfang J, Rocha S, Moergel M, Behrens E, Nicolau P. Platform switch versus platform match in the posterior mandible – 1-year results of a multicenter randomized clinical trial. J Clin Periodontol 2014;41:521-529
- (10)** Rocha S, Wagner W, Wiltfang J, Tondela J, Moergel M, Behrens E, Nicolau P, Messias A, Guerra F. Platform switching vs platform matching: three-year results from a prospective randomized-controlled multicenter study. Short Oral Communication Ref 131, EAO 2015, 25th Annual Scientific Meeting, Stockholm, Sweden
- (11)** CAMLOG and Science. Clinical Studies; Version XJ6164.11/2013:P26-27
- (12)** Becker J, Schwarz F, Kirsch A. Verbesserung der marginalen Knochenadaption durch das neue Promote plus Design. Logo 2006;6: 15-17
- (13)** Rocha S, Wagner W, Wiltfang J, Guerra F, Moergel M, Behrens E, Nicolau P. Platform switching vs platform match: interim results from a prospective randomized-controlled multicenter study. Short Oral Communication Ref 100, EAO 2012, 22th Annual Scientific Meeting, Copenhagen, Denmark
- (14)** Rocha S, Wagner W, Wiltfang J, Guerra F, Moergel M, Behrens E, Nicolau P. Platform switching versus platform matching: two-year results from a prospective randomized-controlled multicenter study. Poster, 5th ICC 2014, Valencia, Spain
- (15)** Wagner W, Moergel M, Rocha S, Nicolau PM, Guerra F. Clinical performance of the CONELOG® SCREW-LINE implant in the posterior mandible - preliminary 2-year results of a prospective two center pilot study. Poster P464, EAO 2014, 24th Annual Scientific Meeting, Rome, Italy
- (16)** Wagner W, Moergel M, Nicolau P, Messias A, Rocha S, Guerra F. Clinical performance of conical tapered and platform-switched implants in the posterior mandible – preliminary 3-year results of a prospective two-center study. Poster P369, EAO 2015, 25th Annual Scientific Meeting, Stockholm, Sweden

CONELOG® is not registered in all markets.

HEADQUARTERS

CAMLOG Biotechnologies AG | Margarethenstrasse 38 | CH-4053 Basel
Tel. +41 61 565 41 00 | Fax +41 61 565 41 01 | info@camlog.com | www.camlog.com

